

Cryosurgery Device

CRYO-S Mini

For dermatosurgery and podiatry



CRYO-S Mini devices for dermatosurgery and podiatry

Cutaneous cryosurgery in the podiatric practice refers to the localized application of freezing temperatures to achieve a controlled destruction or removal of skin lesions on the foot, such as plantar warts and granulation tissue.

Cryo-S Mini is a very efficient and powerful, cryosurgical device that can easily remove most common skin pathologies in just a few seconds: plantar warts, verrucae, skin tags, solar lentigo, pigmented spots, actinic keratoses, seborrheic keratoses, hemangioma, and many more.


Cryo-S Mini, as a very simple, everyday cryosurgical device is ideal for treating benign skin lesions, extremely accurately, quickly, effectively and with no stress or pain.

Freezing medium:

Nitrous suboxide (N₂O) cumulates over two times more energy than liquid nitrogen, therefore 7kg of N₂O is sufficient for approximately 4 hours of operation at any time. N₂O can be stored as long as necessary without losses for evaporation




SD10 Open type cryoprobe (spray). This is only one in the world, patented, spray type cryosurgical probe with fluent, adjustable freezing power.

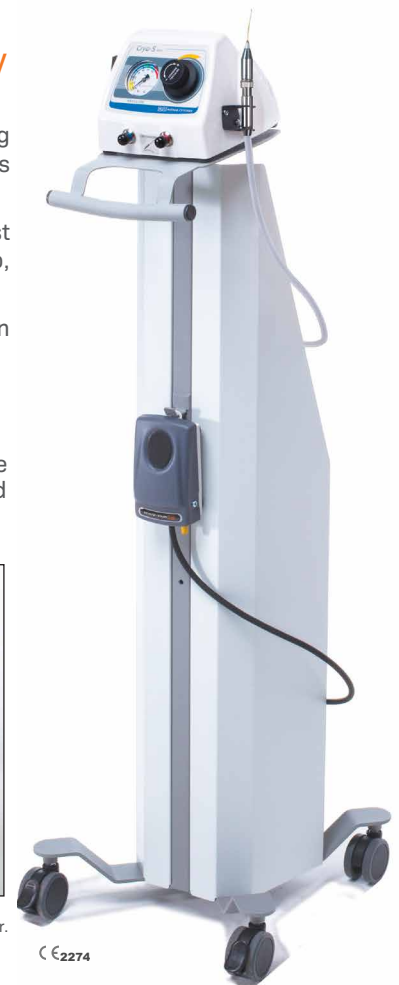

€ 2274

BEFORE

AFTER

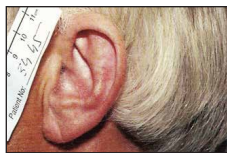

Verruca seborrhoica of auricle



Keratoacanthoma of upper lip



Epithelioma basocellulare of lower eyelid



Verrucae vulgares

Open type cryoprobe SD10 (spray)

Original, patented solution of METRUM CRYOFLEX spray cryoprobe with smooth regulation of freezing power enables effective destruction of changes of any size from very small ones (few mm) to large ones (several cm) by direct spraying of liquid nitrous oxide on the lesion. Using additional freezing surface limiters or cotton tips gives a possibility to shape cryodestruction area very precisely.

Advantages of cryosurgery with CRYO-S MINI in dermatosurgery

- bloodless method
- leaves no scar or scar is "noncolloidal"
- in most cases local anesthesia is not needed
- treatments can be single or multi-stage
- low invasive method acceptable for patients in all ages (most of them can be treated in out-patient clinics or One Day Surgery centers)
- treatment is safe
- excellent cosmetic effects



Contact cryoprobes

€ 2274

METRUM CRYOFLEX

Manufacture
ul. Zielna 29
05-082 Blizne Łaszczyńskiego,
POLAND, EU

Headoffice

ul. Kolejowa 16A
05-092 Łomianki,
POLAND, EU

Tel: +48 22 33 13 750
+48 22 33 13 830
Fax: +48 22 33 13 766

export@metrum.com.pl
sales@metrum.com.pl
www.metrum.com.pl